3M**Quantitative Fit Test Adapter 601****Instructions for use of the 3M™ Quantitative Fit Test Adapter 601**

The 3M™ Quantitative Fit Test Adapter 601 is designed to permit quantitative fit testing with an individual's personal 3M™ 6000, 7000, or FF-400 Series Facepiece with bayonet style cartridge/filter attachments. This is an alternate method to sharing a probed fit test respirator.

Important: Keep these *Replacement Instructions* for reference.

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3M Occupational Health and Environmental Safety Division

3M Center, Building 0235-02-W-70
St. Paul, MN 55144-1000
Printed in U.S.A.

3M is a trademark of 3M Company.
© 2010 3M Company. All rights reserved.

**3M Occupational Health and Environmental Safety Division
3M Canada Company**

P.O. Box 5757 London, Ontario N6A 4T1
Printed in U.S.A.

3M is a trademark of 3M Company.
©2010 3M Company. All rights reserved.

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98-0060-0176-6
34-8704-8530-6

3M™ Quantitative Fit Test Adapter 601 Installation

1. Remove the filter from the right side of the respirator. (Fig. 1)
2. Cut a length of sampling tubing as follows;

3M™ 6000 Series Half Facepiece	- 3 inches
3M™ 6000 Series Full Facepiece	- 4 1/4 inches
3M™ 7500 Series Half Facepiece	- 3 inches
3M™ 7800 Series Full Facepiece	- 4 1/4 inches
3M™ FF-400 Series Full Facepiece	- 4 1/4 inches
3. Slide one end of cut sampling tube onto the inside fitting of the 601 Adapter. (Fig. 2)
4. Make certain the 601 Adapter's inhalation valve is in place. (Fig. 3)
5. Insert the other end of the sampling tube through the respirator filter holder and past the respirator's inhalation valve. Inserting the sample tube through the filter holder in the area shown minimizes the chance of kinking tubing during adapter installation. With the full facepiece respirator feed this tubing end also past the nose cup inhalation valve. (Fig. 4)
6. Engage the 601 adapter to the respirator by aligning parts, pushing together and turning adapter clockwise ¼ turn to stop.
7. Attach a suction cup to the forked end of a retainer clip. (Fig. 5)
8. Install the retainer clip and suction cup onto the sampling tube inside the respirator facepiece. (Fig. 6)

9. Fasten the suction cup to the inside surface of the respirator facepiece of full facepiece nose cup. The suction cup should be located between the nose and mouth of the test subject. It may be necessary to moisten the suction cup to make it stick. Make certain the sampling tube is not crimped or kinked. (Fig. 7)
 10. Install a 3M™ P100 filter onto the 601 adapter. Be sure the same filter is also in place on the respirator's other filter holder.
 11. Attach the quantitative measuring device's sample hose to the outside fitting of the 601 adapter. (Fig. 8)
 12. Standard quantitative fit testing procedures can now be followed.
- Note:** Be sure to remove the 601 fit test adapter assembly and make certain the respirator's inhalation valve is in place and functional before allowing the respirator to leave the fit test area.

For more information

In United States, contact:

Website: www.3M.com/OccSafety
Technical Assistance: 1-800-243-4630

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Internet : www.3M.com/CA/OccSafety

Assistance technique/Technical Assistance :

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